

# Ambu® AuraStraight™



## Key Benefits

- Reinforced tip resists folding over during insertion and plugs the upper esophageal sphincter
- Cuff and airway tube molded as single unit for extra safety - no separation
- The surface has an EasyGlide texture to ease insertion
- Extra soft cuff ensures the best possible seal with least possible mucosal pressure
- Pilot balloon identifies mask size and provides precise tactile indication of degree of inflation
- Convenient depth marks for monitoring correct position of the mask
- Color coded pouch with directions for use
- Available in 8 sizes fitting a wide range of patients
- MR safe
- Phthalate-free material

## Ambu® AuraStraight™









Ambu AuraStraight is a disposable laryngeal mask designed for use both in anesthesia and emergency medicine. It provides users with an even greater choice and is an alternative to the already popular AuraOnce curved laryngeal mask.


Unlike other laryngeal masks, AuraStraight features a unique one-piece-mold design, which ensures not only consistency, but also that there are no rough edges on the airway tube or cuff.

Furthermore, the AuraStraight provides an extra soft, thin cuff. Thus, the seal more readily conforms to the shape of the airway with the minimal amount of pressure. Additionally, the reinforced tip and EasyGlide surface aids the ease of insertion and extraction.

Ambu AuraStraight comes in a full range of 8 sizes from neonate to large adult.

## Specifications

	Infant		Pediatric		Adult			
Mask size	#1	#1 1/2	#2	#2½	#3	#4	#5	#6
Packaging colour code								
Patient weight	<5 kg	5-10 kg	10-20 kg	20-30 kg	30-50 kg	50-70 kg	70-100 kg	>100 kg
Maximum cuff inflation volume	4 ml	7 ml	10 ml	14 ml	20 ml	30 ml	40 ml	50 ml
Maximum intra-cuff pressure	60 cm H <sub>2</sub> O							
Airway connector	15 mm ISO 5356-1							
Min. I.D. Tube	5.7 mm	6.5 mm	7.4 mm	8.9 mm	9.8 mm	10.7 mm	11.0 mm	12.1 mm
Max. O.D. Tube	9.6 mm	11 mm	12.2 mm	14.4 mm	16.1 mm	17.6 mm	19.3 mm	19.5 mm
Inflation valve	Luer cone ISO 594-1							
Appropriate storage temperature	+10° C - +25° C							
Weight	9,8 g	13,6 g	17,8 g	27,0 g	38,2 g	44,9 g	62,9 g	69,6 g
Internal volume of ventilatory pathway	3.5 ml	5.4 ml	6.7 ml	9.7 ml	13 ml	15 ml	20 ml	22 ml
Pressure drop	<1.5 cm H <sub>2</sub> O at 15 l/min	<1.0 cm H <sub>2</sub> O at 15 l/min	<1.9 cm H <sub>2</sub> O at 30 l/min	<1.0 cm H <sub>2</sub> O at 30 l/min	<2.2 cm H <sub>2</sub> O at 60 l/min	<1.8 cm H <sub>2</sub> O at 60 l/min	<1.2 cm H <sub>2</sub> O at 60 l/min	<1.2 cm H <sub>2</sub> O at 60 l/min
Min. interdental gap	12.8 mm	15.1 mm	17.1 mm	20 mm	23 mm	25.4 mm	27.4 mm	28.7 mm
Internal Pathway	10.9 cm	11.6 cm	14.4 cm	15.9 cm	16 mm	17.1 mm	18.8 mm	20.4 cm
<b>Specialized use</b>								
Max. FOB size	2.7 mm	2.7 mm	3.9 mm	3.9 mm	5.4 mm	5.4 mm	5.4 mm	5.4 mm

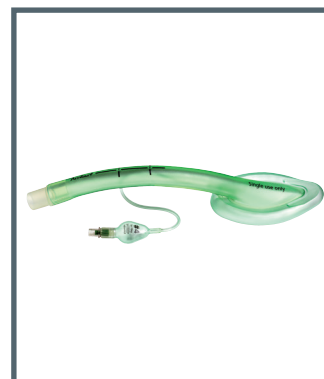

AuraStraight



AuraStraight with reinforced tip

## Materials

Product	Phthalate-free	Material
Airway tube	Yes	PVC
Cuff	Yes	PVC
Airway connector	Yes	PCTG
Pilot Balloon	Yes	PVC
Inflation valve	Yes	PVC/Silicone
Pilot tube	Yes	PVC

The Ambu AuraStraight is not made with natural rubber latex

Packaging	Quantity	Material
Pouch	1	Tyvek®/Polyethylene Terephthalate (PET)/Polyethylene (PE)
Box	10	Cardboard

## Ordering specifications

Item no.	Size	Type
324 100 000	1	Disposable Laryngeal Mask - Straight
324 150 000	1 1/2	Disposable Laryngeal Mask - Straight
324 200 000	2	Disposable Laryngeal Mask - Straight
324 250 000	2 1/2	Disposable Laryngeal Mask - Straight
324 300 000	3	Disposable Laryngeal Mask - Straight
324 400 000	4	Disposable Laryngeal Mask - Straight
324 500 000	5	Disposable Laryngeal Mask - Straight
324 600 000	6	Disposable Laryngeal Mask - Straight



AuraStraight in colour-coded pouch with instructions for use

### Ambu A/S

Baltorpbakken 13  
DK - 2750 Ballerup  
Denmark  
Tel. +45 7225 2000  
Fax +45 7225 2053  
www.ambu.com